


Airway-Circulation-Evaluator

READY WHEN YOU NEED IT!
Resuscitation ACE may be kept on the resuscitation bag, ready to use at a moments notice.

PROOF WHEN YOU NEED IT!
The Resuscitation ACE color change indicates presence of CO₂.

ECONOMICAL WHEN YOU NEED IT
The simplicity of the Resuscitation ACE design also means that it's cost efficient.

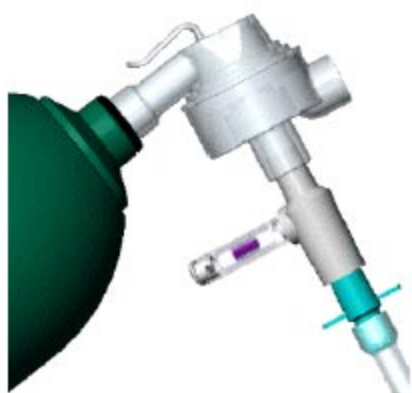

"First blush" color (after 6 breaths) helps confirm you have successful intubation.



Resuscitation ACE works reliably during CPR (low circulation). Light purple indicates CO₂ level of approx. 0.5%.



Resuscitation ACE gives you a semiquantitative circulation check. Dark purple indicates CO₂ level greater than 1.5%.

US Patent 4945918
And others pending.

Of course, a CO₂ monitor is recommended in the Advanced Cardiac Life Support (ACLS) guidelines from the AHA. Using one is simply a smart adjunct to clinical assessment.

To be of up most value, EMS professionals agree that a CO₂ monitor must be an integral part of your emergency response-and that means it must be an integral part of your gear. It can't be an add-on at a critical moment, certainly not an afterthought. Obviously that means it must be designed specifically for demanding EMS use.

Only the *Resuscitation ACE* meets that standard. Try putting any CO₂ monitor to the test-start with an actual training situation. How fast can you put it to use? Must you first remove it from packaging? Does it require extra steps? What could go wrong? Will it provide clear assessment?

There's a bottom line to all of this and the choice is not difficult. You owe it to yourself to put the Resuscitation ACE CO₂ monitor to the test.

HERE'S WHY *RESUSCITATION ACE* DESERVES A CLOSER LOOK:

Easy to Use

The Resuscitation ACE can stay on the bag, not in a foil package or med kit. The unique crush-capsule design makes it possible to activate the CO₂ indicator in an instant after you attach the bag to the E-T tube. Suction is possible because there are no right angles in the direct flow design. No right angles can result in easier bagging of the patient.

PROOF

The indicator delivers a clear, simple assessment of airway and circulation for confident patient monitoring in emergency situations.

Ordering Information

Cat. #: RA100 ...10 per case

* Resuscitation ACE is also available packaged with VENTLAB resuscitation bags.

Resuscitation ACE Inc.
155 Boyce Drive
Mocksville, N.C. 27028

Toll-Free: 800-593-4654
Phone: 336-753-5000
Fax: 336-753-5002